

FULL M-FINGER

CHECK FIT INSTRUCTIONS

INLCUDED IN PACKAGE

- 1 Outer pre-preg frame
- 1 Inner silicone liner
- Full M-Finger Set

FITTING INSTRUCTIONS

- Don silicone and carbon frame to assess fit.
- If the liner and socket fits well, proceed to mounting.
- If socket does **NOT** fit, please mark all adjustments necessary and return to the address below for new fabrication. **PLEASE CONTACT US BEFORE PROCEEDING.**

MOUNTING

- Mark proper alignment for M-Finger set.
- Apply a small amount of 60 second epoxy to the mounting base plate and hold in place until set.
- USE THE MINIMUM AMOUNT OF EPOXY NEEDED TO HOLD ALIGNMENT.
- DO NOT COVER SCREW HOLES WITH EPOXY.
- Once set, please return to the address below for final fabrication. Please include any additional notes necessary.

RETURN FITTING MATERIALS FOR FABRICATION TO:

Hanger Fabrication Network
Attn: Silicone Interfacing Department
10 County Line Drive, Unit E
Cromwell CT 064416
Phone: 860-667-5357 Fax: 860-632-0013