

TITAN FINGER

CHECK FIT INSTRUCTIONS

INLCUDED IN PACKAGE

- Titan finger with base
- Base screw is covered with silicone putty for protection, **do not remove**
- Custom designed silicone and carbon frame

FITTING INSTRUCTIONS

- Don silicone and carbon frame to assess fit.
- If the liner and socket fits well, proceed to mounting.
- If socket does **NOT** fit, please mark all adjustments necessary and return to the address below for new fabrication. **PLEASE CONTACT US BEFORE PROCEEDING.**

MOUNTING

- Gently bend tabs on titan finger base to fit the shape of the carbon frame.
- Mark proper alignment of each finger.
- Apply a minimal amount of 60 second epoxy to tabs only, then hold in place until set. DO NOT COVER SCREW HEAD ON BOTTOM OF FINGER.
- Once set, please return to the address below for final fabrication.
Please include any additional notes necessary.

RETURN MATERIALS FOR FABRICATION TO:

Hanger Fabrication Network
Attn: Silicone Interfacing Department
10 County Line Drive, Unit E
Cromwell CT 064416
Phone: 860-667-5357 Fax: 860-632-0013