

FULL M-FINGER

DELIVERY INSTRUCTIONS

INLCUDED IN PACKAGE

- Definitive Device
- Spare Parts

FITTING INSTRUCTIONS

- Don device on patient.
- Check fit and alignment.
- Adjust tension of cable for optimal patient use.
- Once length of cable is set, tie a knot and seal it with glue.
- Insert set screw to hold cable in place.
See reference materials for further details

CARE & MAINTENANCE

- New users should start with a wearing schedule and build up to a full day of wear.
- Patients should clean the silicone liner with soap and water or hand sanitizer daily.
- Do not expose the device to corrosive or caustic materials, UV rays (such as tanning beds) or temperature extremes.
See reference materials for assembly/disassembly instructions.**

CONTACT INFORMATION

Hanger Fabrication Network
Attn: Partial Hand Department
10 County Line Drive, Unit E
Cromwell CT 064416

Phone: 860-667-5357
Fax: 860-632-0013